





Prepare the skin according to the standard hospital protocol for dressing application. Skin prep or hair removal may be required on some patients for better adhesion.

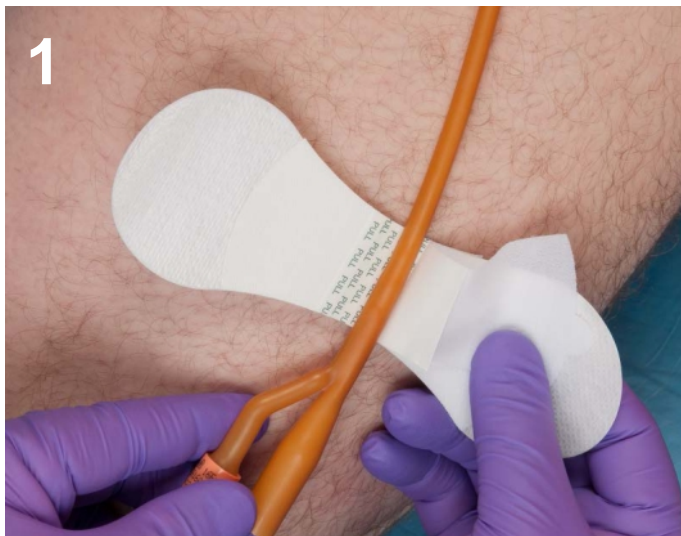

Pull the paper backing from one side of the Grip-Lok®. Then pull the paper backing from the other side to secure the Grip-Lok® in the desired position on the skin.



Open the top fabric over-strap and remove the paper liner to expose the adhesive strips.

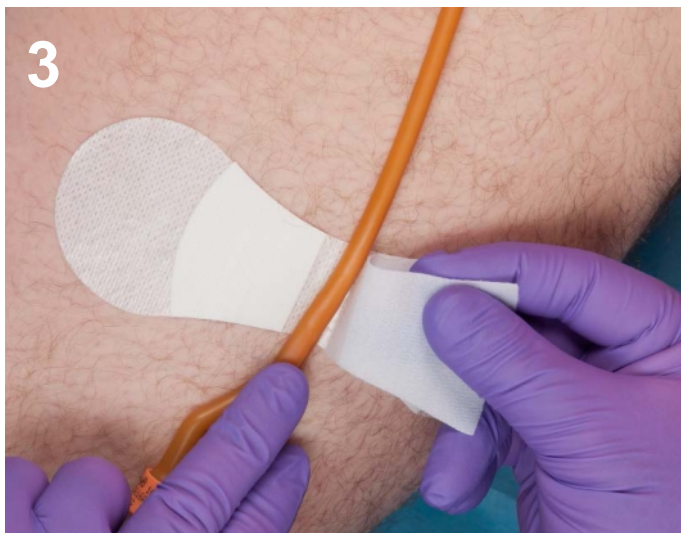

Place the catheter centrally on the bottom adhesive area.



Secure the fabric over-strap over the catheter so that it is placed between the adhesive areas, then close the top over-strap.