




Prepare the skin according to the standard hospital protocol for dressing application.

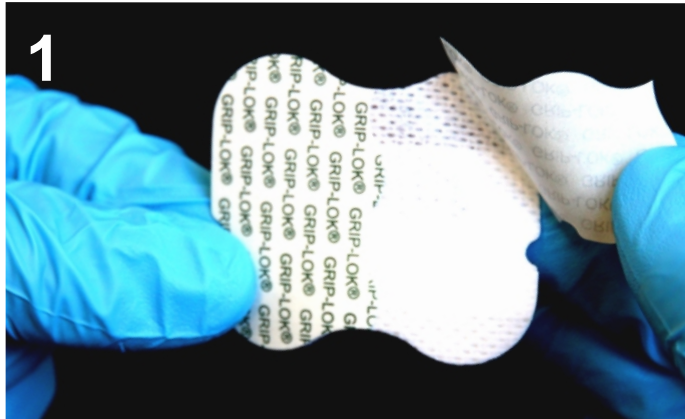


With the Grip-Lok® logo facing you, peel back and remove both sides of the adhesive liner.

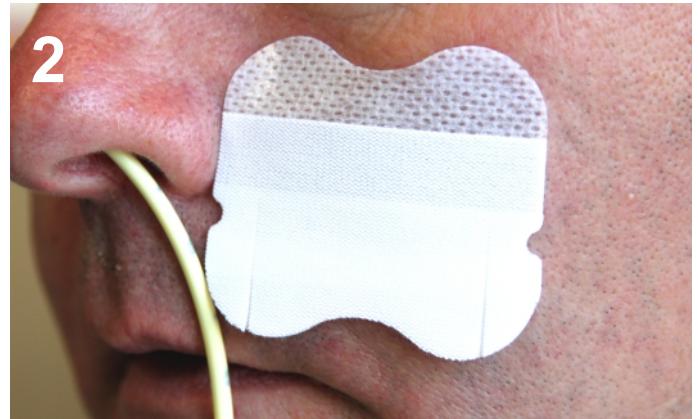


Place next to nostril with the side notch positioned slightly below the nostril with the lid facing downwards. Gently apply pressure to adhere to skin.

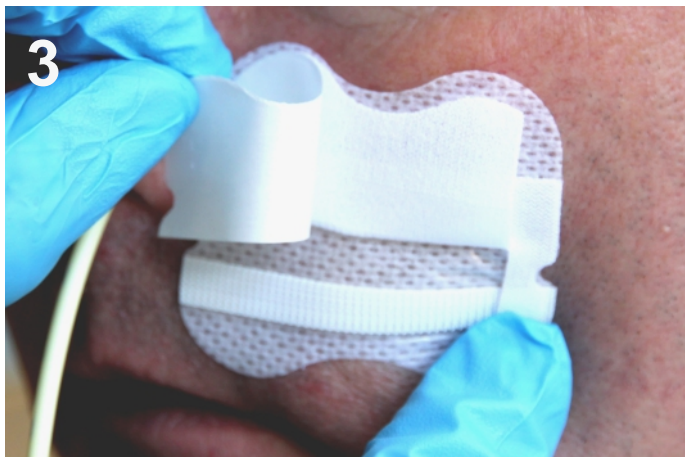


Lift the lid up and remove the release liner revealing a 3-section lid. Close the small section closest to the ear.

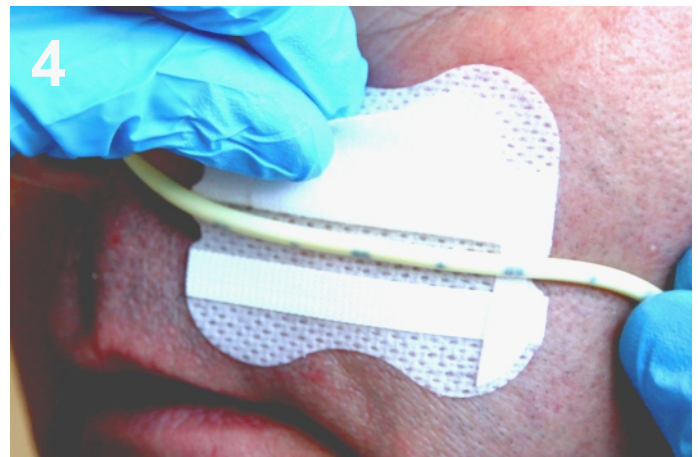

Position the nasal gastric tube centrally along the adhesive strip.

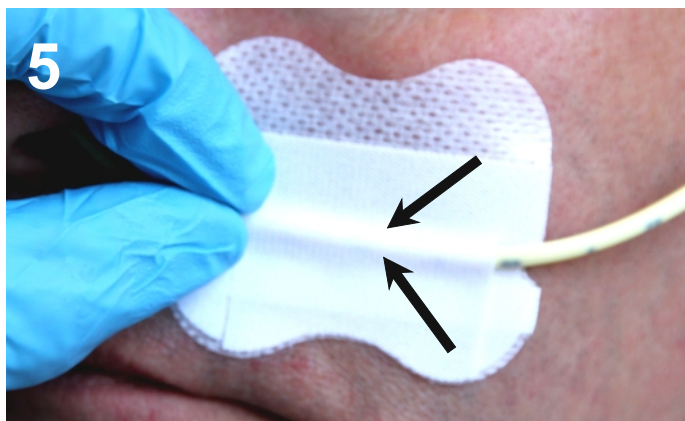

Close the remaining two sections of lid around the nasal gastric tube making sure to pinch around and along the tube.

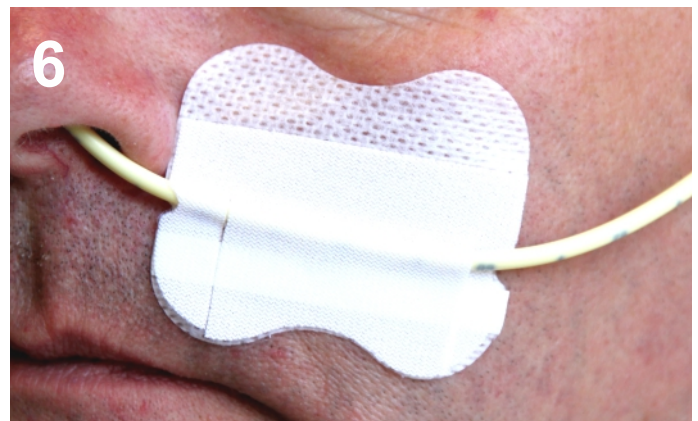

Grip-Lok® applied with nasal gastric tube secured.